**Instructions
for use**
English

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Audífono

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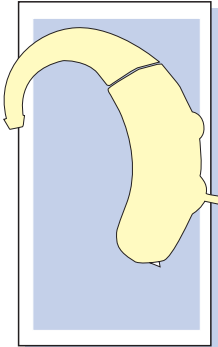
SIEMENS

Behind-the-ear hearing instruments Instructions for use

This Quick Guide of user instructions describes how to operate your hearing instruments quickly and easily.

It also provides you with important information on how to handle your instruments and batteries.

Additional tips are given in the User Guide (not supplied with every instrument).

Your hearing instrument specialist can answer any further questions about instructions and proper use.

E

Safety notes for batteries

WARNING.

Keep batteries and instruments out of the reach of children.
Batteries can be harmful if swallowed.
If swallowed consult a physician or a hospital immediately.

CAUTION.

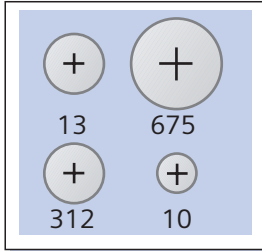
Leaking batteries damage the instrument.
Remove the batteries when the instrument is not in use for
a prolonged period of time.

NOTE.

Do not throw used batteries into household trash.
Dispose them according to national regulations or return
them to your hearing instrument specialist.

Inserting or changing a battery


Battery type
original size

NOTE.

In some instruments the sound may become fainter or you may hear additional alerting signals when the battery power is low. If this happens, change the battery.

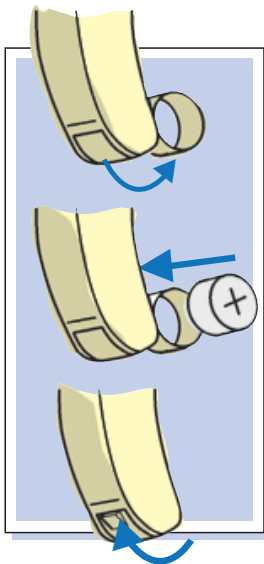E

Open the battery compartment.

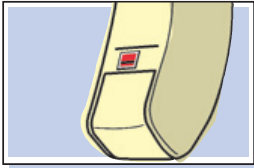
Insert the battery.
Make sure the battery and compartment symbols are aligned.

Close the battery compartment.

Lock for the battery compartment

AVAILABLE ON SELECT MODELS


To lock or unlock, move the slide with a suitable pointed tool.


Unlock

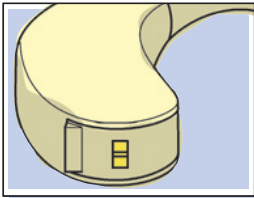lock

OR

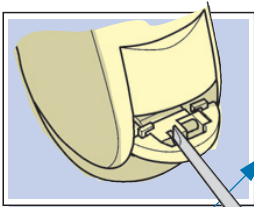To lock or unlock, move the slide with a suitable pointed tool.

Partially open the battery compartment.

lock

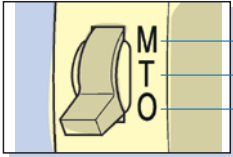
unlock


Turning your hearing instrument on/off


E

- On
- Special function
- Off

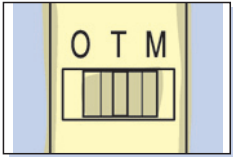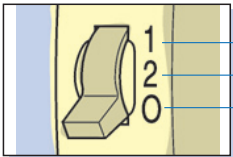- OR
Some instruments have a horizontal program button


- OR
1st program (On)
2nd program
Off


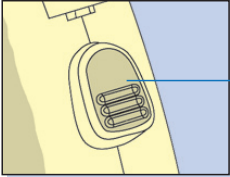
Turning your hearing instrument on/off

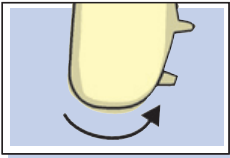OR
Program button:
Press and hold the button for 3 seconds

OR
ON:
Close the battery compartment

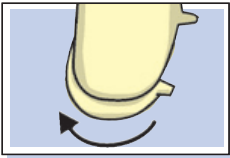
OFF:
Partially open the battery compartment


Adjusting the volume

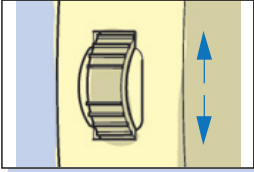
E

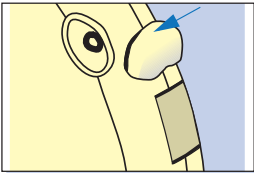Increase volume:
turn wheel up
Decrease volume:
rotate wheel down

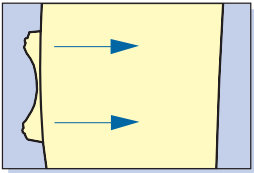OR
Increase volume:
press upward
Decrease volume:
press downward

OR
Increase volume:
press up
Decrease volume:
press down

IMPORTANT

If operated incorrectly, hearing instruments with high amplification can cause permanent hearing damage. Please ask your hearing instrument specialist.

NOTE

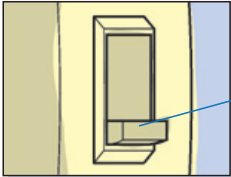
For instruments with automatic volume control, the volume will adjust automatically when in use!


Selecting the hearing program

AVAILABLE ON SELECT MODELS



EITHER

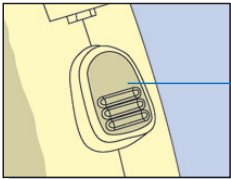Change program:
Flip the switch


OR

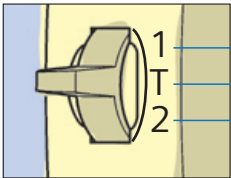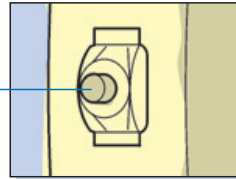Press
the button

OR

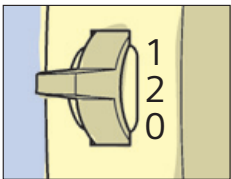1st Hearing program
 T Telephone coil
 2nd Hearing program

OR

1st Hearing program
 2nd Hearing program
 Flip the toggle switch

Connecting Audio Input to external devices

AVAILABLE ON SELECT MODELS

E

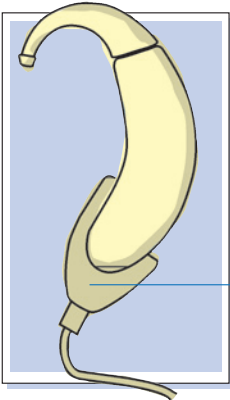
With certain hearing instruments it is possible to connect accessories through an audio shoe.

Ask your hearing instrument specialist for further details.

WARNING.

Danger of electrical shock!

Connect the audio input only to a line-operated device which conforms with the safety requirements of IEC 65.

Available on select models

Audio shoe

Using the telephone

AVAILABLE ON SELECT MODELS

With the AutoPhone functionality, your hearing system will automatically switch to the telephone program when you hold a hearing-instrument-compatible-telephone headset to your hearing instrument. After telephone use it will automatically return to the previously used hearing program.

Most types of telephones require the small **AutoPhone magnet** to produce a magnetic field strong enough to activate the AutoPhone function.

Your hearing instrument specialist will program your hearing system to your individual requirements.

NOTE

See the AutoPhone Magnet User Guide for instructions on how to position the magnet. Use only the magnet that is delivered with the hearing instrument.

Magnets can pose risks to electronics and magnetic storage mediums when placed in close proximity. Take care not to set magnets near computers, color monitors, television sets, floppy discs, video/audio tape and other electronic equipment.

Keep magnets away from the reach of children. If swallowed consult your physician.

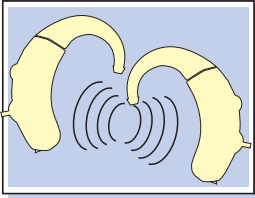
Magnets may interfere with the operation of life supporting systems (i.e. pace-makers, magnetic valves etc).

Use magnets only if not in doubt about the safe distance.

The Safe distance between a pacemaker and magnet should be at least 10 cm (4 inch).

e2e wireless™

Unique to particular hearing systems is the e2e wireless™ technology.

E

The e2e wireless functionality:

- Automatically and simultaneously adjusts the systems to changes in listening situations.
- Provides synchronization of the settings in the system. If you change the volume or program on one instrument, the other is automatically changed.
- Allows use of wireless accessories including the ePocket™ (remote control).

e2e wireless™

Your hearing instrument specialist can activate or deactivate the e2e wireless functionality of your hearing system.

The ePocket™ can be activated or deactivated separately. Consult your hearing instrument specialist for instructions on use of this feature with your particular hearing system.

ATTENTION:

The performance of the e2e wireless could be affected by electro-magnetic interference, such as a computer monitor or halogen lamp systems controlled by a switching power supply. Move away from the source of interference if you experience difficulty.

Operating the hearing instrument in T-coil mode the wireless function may cause a pulsing noise. This is due to the pick-up of the transmission signal by the hearing aid T-coil. When your hearing instrument is in T-coil mode please use the remote control only with a distance greater than 10 cm.

All ePockets operate on the same frequency. Although it is very unlikely, another ePocket within operating distance of less than 1 meter (40 inches) could potentially change the settings of your hearing system. Contact your hearing instrument specialist if this occurs. Your ePocket can be sent to the manufacturer to address this issue if necessary.

General notes on safety

E

ATTENTION.

Protect against dampness, moisture, overheating and physical damage.

ATTENTION.

Remove your hearing instrument before:

- showering/bathing and when using hairspray
- coming into contact with strong magnetic fields (e.g. medical devices)
- undergoing an X-ray examination

WARNING.

Do not use your hearing instrument in explosion hazard areas!

In case of instrument failures

- Is the instrument turned on?
- Is the battery inserted correctly?
- Is the battery power low?
- Is the battery compartment closed completely?
- Is the volume control turned up?
- Is the sound opening in the ear mould clogged with wax?

NOTE ON PROPER CARE.

Clean the ear mold regularly to remove earwax buildup.
Consult with your hearing instrument specialist for further
instructions on cleaning your particular instrument.

Disposal information

Within the European Union, the marked equipment is covered by "Directive 2002/96/EC of the European Parliament and of the Council of 27 January 2003 on waste electrical and electronic equipment. Amended by Directive 2003/108/EC" (WEEE).

E

Recycle your hearing instrument according to national regulations.

For disposal within the EU please send device to the following address:

United Kingdom

Siemens
Hearing Instruments
Newton Road
Crawley RH109TT

Malta

Charles de Giorgio LTD
39-40 South Street
Valetta CMR 01
P.O.Box 374

Ireland

Howth Junction
Business Center
Kilbarrack
5 Dublin

www.hearing-siemens.com

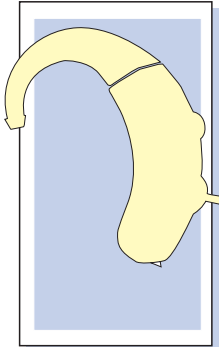
With the CE marking Siemens confirms compliance with the European Directive 93/42/EEC concerning medical devices.

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Siemens Hearing Instruments Pty Ltd
Siemens Medical Instruments Pty Ltd