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**Instructions
for use**
English

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**Hörgeräte
Hearing instruments
Intra-auriculaires
Apparecchi acustici
Hoortoestel**

SIEMENS

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Kunde: _____

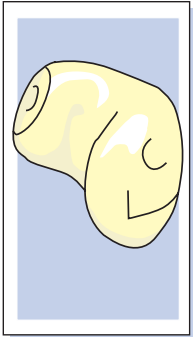
Datum: _____

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In-the-ear hearing instruments Instructions for use

This Quick Guide of user instructions describes how to operate your hearing instruments quickly and easily.

It also provides you with important information on how to handle your instruments and batteries.

Additional tips are given in the User Guide (not supplied with every instrument).

Your hearing instrument specialist can answer any further questions about instructions and proper use.

E

Safety notes for batteries

WARNING.

Keep batteries away from children!
Batteries can be harmful if swallowed.
If swallowed consult a physician or a hospital immediately.

ATTENTION.

Remove the batteries when the instrument
is not in use for a prolonged period of time.
Replace leaking batteries without delay.

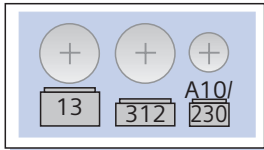
NOTE.

Do not throw away your dead batteries!
Return them to your hearing instrument specialist
for proper, environmentally conscious disposal.

Inserting or changing a battery


Battery type
original size

NOTE.

In some types of instruments the sounds may become fainter or you may hear additional sounds.

This means that the battery power is low. If this happens, change the battery.

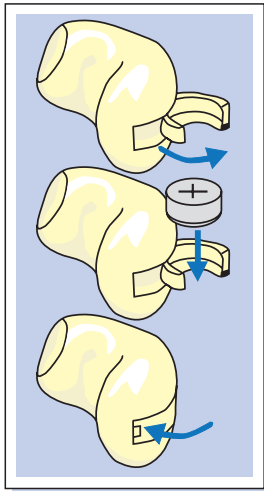E


Open the battery compartment.

Insert the battery.
Make sure the "+" symbols on the battery and compartment are aligned.

Close the battery compartment.

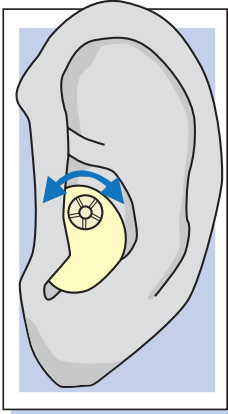Adjusting the volume

INSTRUMENTS WITH VOLUME CONTROL

LOUDER:

Rotate the wheel forward

QUIETER:

Rotate the wheel back

INSTRUMENTS WITH AUTOMATIC VOLUME CONTROL

The volume is adjusted automatically when in use.

IMPORTANT.

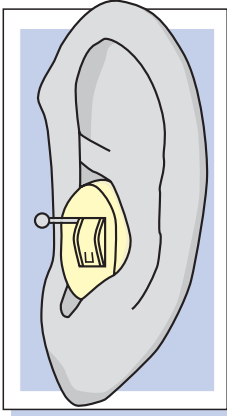
If operated incorrectly, hearing instruments with high amplification can cause permanent hearing damage.

Please ask your hearing instrument specialist.


Turning your hearing instrument on/off

BATTERY COMPARTMENT:

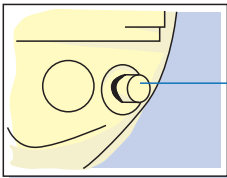Fully open the battery compartment to turn your hearing instrument off; fully close the compartment to turn it on.


OR

PROGRAM BUTTON:

(Available on select models)

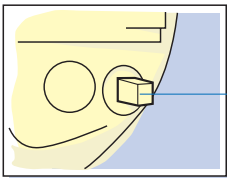Press and hold the button for 3 seconds to turn the instrument on or off.

Turning on:

Flip the switch forward.

Turning off:

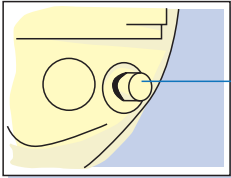
Flip the switch backward.


Selecting the hearing program/telecoil

AVAILABLE ON SELECT MODELS

Press the button to change the program.

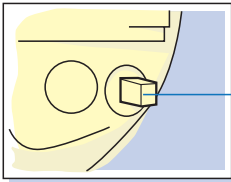OR

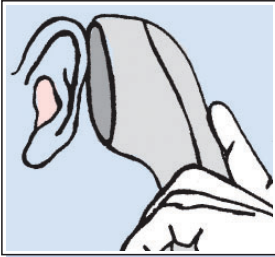
Switching on:
toggle the switch

Using the telephone

AVAILABLE ON SELECT MODELS

With the AutoPhone functionality, your hearing system will automatically switch to the telephone program when you hold a hearing-instrument-compatible-telephone headset to your hearing instrument.

After telephone use it will automatically return to the previously used hearing program.

E

Most types of telephones require the small **AutoPhone magnet** to produce a magnetic field strong enough to activate the AutoPhone function.

Your hearing instrument specialist will program your hearing system to your individual requirements.

NOTE:

Magnets can pose risks to electronics and magnetic storage mediums when placed in close proximity. Take care not to set magnets near computers, color monitors, television sets, floppy discs, video/audio tape and other electronic equipment.

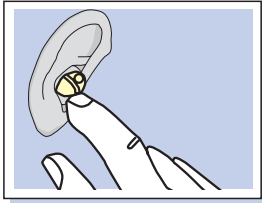
Keep magnets away from the reach of children. If swallowed consult your physician.

Magnets may interfere with the operation of life supporting systems (i.e. pace-makers, magnetic valves etc).

Use magnets only if not in doubt about the safe distance. The Safe distance between a pacemaker and magnet should be at least 10 cm (4 inch).

Removing the instrument from your ear

To remove the hearing instrument, push lightly on the back of your ear to help loosen it or pull gently on your ear.

Then using your index finger and thumb, slowly pull the hearing instrument from your ear.

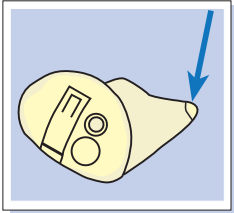
CAUTION:

Never pull on the battery compartment door or volume control wheel to remove the hearing instrument, as this could cause damage

Instruments with wax guard cap

Your hearing system may be equipped with a sieve, recess or guard on the sound outlet to prevent earwax from entering the hearing system.

Your hearing care specialist will provide you with the wax protection appropriate for your system and can advise you how to clean or replace it.

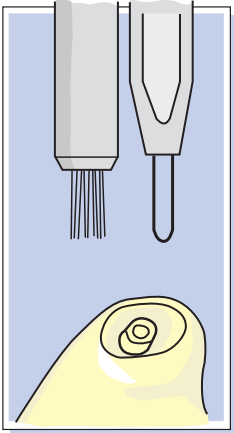
E

ATTENTION.

Never clean the cap as it may cause damage to the wax guard.

Instruments with wax guard


Remove earwax daily
with loop or brush

NOTE.

If your instrument should have a different wax guard, your hearing instrument specialist will instruct you on its use.

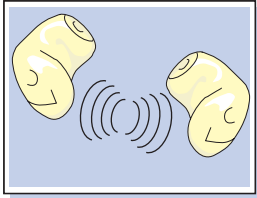e2e wireless™

Unique to particular hearing systems is the e2e wireless™ technology.

The e2e wireless functionality:

- Automatically and simultaneously adjusts the systems to changes in listening situations.
- Provides synchronization of the settings in the system. If you change the volume or program on one instrument, the other is automatically changed.
- Allows use of wireless accessories including the ePocket™ (remote control).

E

e2e wireless™

Your hearing instrument specialist can activate or deactivate the e2e wireless functionality of your hearing system.

The ePocket™ can be activated or deactivated separately. Consult your hearing instrument specialist for instructions on use of this feature with your particular hearing system.

ATTENTION:

The performance of the e2e wireless could be affected by electro-magnetic interference, such as a computer monitor or halogen lamp systems controlled by a switching power supply. Move away from the source of interference if you experience difficulty.

All ePockets operate on the same frequency. Although it is very unlikely, another ePocket within operating distance of less than 1 meter (40 inches) could potentially change the settings of your hearing system. Contact your hearing instrument specialist if this occurs. Your ePocket can be sent to the manufacturer to address this issue if necessary.

General notes on safety

ATTENTION.

Protect against dampness, moisture, overheating and physical damage.

ATTENTION.

Remove your hearing instrument before:

- showering/bathing and when using hairspray
- e.g. in the practice or hospital: coming into contact with strong magnetic fields or high-frequency fields
- undergoing an X-ray examination

E

WARNING.

Do not use your hearing instrument in explosion hazard areas!

In case of instrument failures

- Is the battery inserted correctly?
- Is the instrument switched on or the battery compartment closed?
- Is the battery power low?
- Is the volume control turned up?
- Is the instrument clogged by earwax?

NOTE.

Choose your instrument care accessories to your needs. Your hearing instrument specialist will advise you and assist you in their use.

Within the European Union, the marked equipment is covered by "Directive 2002/96/EC of the European Parliament and of the Council of 27 January 2003 on waste electrical and electronic equipment. Amended by Directive 2003/108/EC" (WEEE).

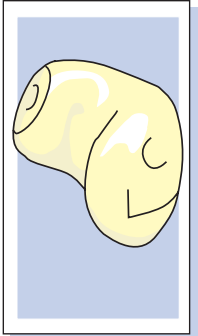
With the CE marking Siemens confirms compliance with the European Directive 93/42/EEC concerning medical devices.

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Siemens Hearing Instruments Pty Ltd
Siemens Medical Instruments Pty Ltd