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Instructions for use
English

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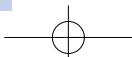
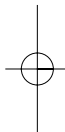
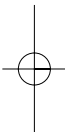
Life Products

Hörgeräte
Hearing instruments
Contours d'oreille
Apparecchi acustici
Hoortoestel

SIEMENS

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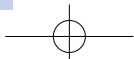
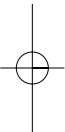
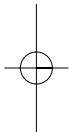
Behind-the-Ear hearing system

Instructions for use

This Quick Guide of user instructions describes how to operate your hearing instruments quickly and easily.

It also provides you with important information on how to handle your instruments and batteries.

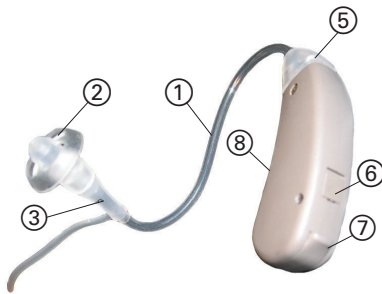
Additional tips are given in the User Guide (not supplied with every instrument).

Your hearing instrument specialist can answer any further questions about instructions and proper use.

E

Your Life BTE (behind-the-ear) hearing system

Life

Optional: Standard earhook

Hearing system for mild to moderate hearing loss

Features

- | | |
|-----------------------|--|
| 1. LifeTube | 6. Cover for programming socket |
| 2. LifeTip | 7. Battery compartment |
| 3. LifeTip Adapter | 8. Color-coded typeplate
red = right instrument
blue = left instrument |
| 4. Earhook (optional) | |
| 5. Microphone opening | |

Note:

- Wear the LifeTube always with the LifeTip.
- Make sure that the LifeTip is completely attached to the LifeTip Adapter.
- Change the LifeTube or LifeTip when they are cracked.
- Put the LifeTube with the LifeTip carefully and not too deep into the ear.
- Use the LifeTube and LifeTip only in the way described in this user guide.

Safety notes for batteries

Warning:

Keep batteries away from children!

Batteries can be harmful if swallowed.

If swallowed consult a physician or a hospital immediately.

E

Attention:

Remove the batteries when the instrument is not in use for a prolonged period of time.

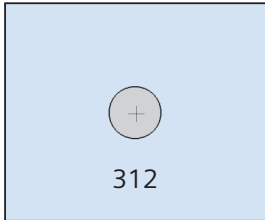
Replace leaking batteries without delay.

Note:

Do not throw away your dead batteries!

Return them to your hearing instrument specialist for proper, environmentally conscious disposal.

Inserting or changing a battery


Battery type
original size

Note:

The sound may become fainter or you may hear additional alerting signals when the battery power is low.

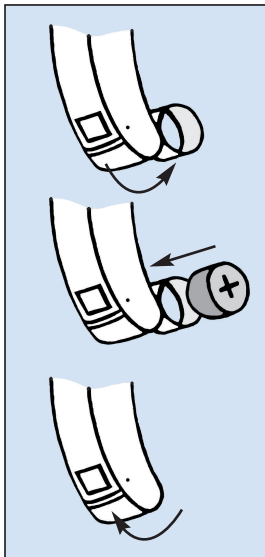If this happens, change the battery.


Open the battery compartment.

Insert the battery. Make sure the battery and compartment symbols are aligned.

Close the battery compartment.

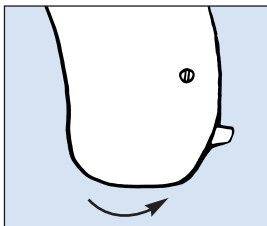
Turning your Behind-the-Ear hearing system on/off

On:

Close the battery compartment

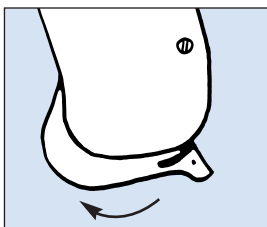E


Off:

Partially open the battery compartment

e2e wireless™

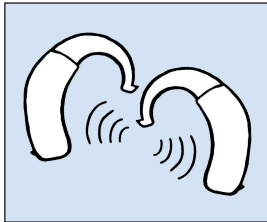AVAILABLE ON SELECT MODELS

Unique to particular hearing systems is the e2e wireless™ technology.

The e2e wireless functionality:

- Automatically and simultaneously adjusts the systems to changes in listening situations.
- Provides synchronization of the settings in the system.
- If you want to change the volume or program on the instrument, it is only possible with the ePocket (remote control).
- Allows use of wireless accessories including the ePocket™ (remote control).

e2e wireless™

AVAILABLE ON SELECT MODELS

Your hearing instrument specialist can activate or deactivate the e2e wireless functionality of your hearing system.

The ePocket™ can be activated or deactivated separately.

Consult your hearing instrument specialist for instructions on use of this feature with your particular hearing system.

E

Attention:

The performance of the e2e wireless could be affected by electro-magnetic interference, such as a computer monitor or halogen lamp systems controlled by a switching power supply. Move away from the source of interference if you experience difficulty.

General notes on safety

Attention:

Protect against dampness, moisture, overheating and physical damage.

Attention:

Remove your hearing instrument before:

- showering/bathing and when using hairspray
- coming into contact with strong magnetic fields (e.g. medical devices)
- undergoing an X-ray examination

Warning:

Do not use your hearing instrument in explosion hazard areas!

In case of instrument failures

- Is the instrument turned on?
- Is the battery inserted correctly?
- Is the battery power low?
- Is the battery compartment closed completely?
- Is the volume control turned up?
- Is the sound opening in the LifeTube or earmold clogged with wax?

E

Note on proper care:

Consult with your hearing instrument specialist for further instructions on cleaning your particular instrument.

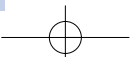
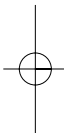
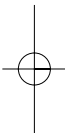
Within the European Union, the marked equipment is covered by "Directive 2002/96/EC of the European Parliament and of the Council of 27 January 2003 on waste electrical and electrical equipment. Amended by Directive 2003/108/EC" (WEEE).

With the CE marking Siemens confirms compliance with the European Directive 93/42/EEC concerning medical devices.

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Siemens Hearing Instruments Pty Ltd
Siemens Medical Instruments Pty Ltd